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EN

Warnings and safety information

All information regarding the use of the product can be found in the instruction manual. Before using it, read its contents and follow the instructions contained therein.

Also read the following information before use:

Warnings for use

Risk of product damage:

- Do not use the otoscope in high humidity or dusty conditions if the device is not properly protected.
- Avoid soaking the otoscope in water unless the manufacturer indicates otherwise in the specifications. Use a slightly damp cloth to clean if necessary.
- Avoid sudden movements or pressure when using the device to avoid damaging delicate mechanical and electronic components.
- Regularly check the condition of the device and battery - do not use a damaged otoscope or charging cables with visible signs of wear.

Risk to the user:

- The device should not be used on damaged tissue, wounds or inflammation of the ear.
- The device should only be used by or under the supervision of adults, especially for children.
- Use only original tips dedicated by the manufacturer to ensure safety and hygiene.
- In case of discomfort or pain during use, stop using the device and consult a doctor.
- For app-enabled devices, make sure you use a secure Wi-Fi network to avoid the dangers of unauthorized access.

Information on proper use

Recommendations for use:

- Make sure the device is fully charged or powered according to the manufacturer's instructions before use.
- Connect the otoscope to the manufacturer's mobile app by following the instructions. Make sure your smartphone is connected to the device via the otoscope's Wi-Fi network.
- Use dedicated soft material tips to gently insert the camera into the ear, avoiding pressure on the walls of the ear canal.
- Avoid using the device for long periods of time in one go - regular breaks reduce the risk of irritation.
- Hold the device steady during use and gently insert the tip into the ear to avoid damage to the internal structures of the ear.
- Clean the tips after each use according to the manufacturer's recommendations.
- Keep the app software updated to take advantage of the latest features and ensure optimal data security.

Maintenance and storage:

- Clean the device after use and store it in a dry, clean place away from direct sunlight and high temperatures.
- Regularly check the condition of the device's terminals, cables and housing for mechanical damage.
- For battery-powered devices, avoid discharging them completely and storing them in extreme temperatures.

Additional precautions

Adaptation to the user:

- Make sure the device has been calibrated and is fully operational before first use.
- For children, the device should only be used under adult supervision and the tips adjusted to the child's sensitive skin.

Damage and service:

- If there are problems with the camera's operation, image distortion, Wi-Fi interruption or visible mechanical damage (e.g., broken tips, damage to the housing), discontinue use and contact the manufacturer's authorized service center.

- Use only original components approved by the manufacturer for repairs and replacement of parts to ensure safety and maintain the warranty.

Statement of compliance

The products have been designed in accordance with the safety requirements of Regulation (EU) 2023/988. They meet the relevant technical standards, including electromagnetic compatibility, safety of use of materials, and resistance to mechanical and electronic damage.

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